

Endo-Safier™

SINGLE USE POLYP TRAP

*Instructions for Use.
Read carefully prior to use.*

Suzhou GZM Medical Co., Ltd.
Room1612, Building 8, Shi Shan Tian Jie Living Plaza,
215011, Suzhou, Jiangsu,
PEOPLE'S REPUBLIC OF CHINA
Tel +86-512-65575235
e-mail: sales@gzmmedical.com
www.gzmmedical.com
Made in China

<u>Reference Number</u>	<u>Content</u>
G-PLP01	Polyp trap, non-sterile, Single Chamber

Description
Polyp trap

FR piège à polype/ DE Polypenfalle ES Trampa de pólipos/ NL Poliepvul/ PT Armadilha de pólipos/ IT Polipo trappola/ DK Polypefælde/ GR Παγίδα Polyp/ SE Polyp fållahU Polipcsapda/ PL Pułapka polipowa/EE Polüüptide lõks HR zamka za polip / CZ Polypová past/ LT Polipų spąstai SI Past za polip

INTENDED USE

The single use Polyp Trap is used for suction retrieval and transportation of endoscopically removed polyps for histological examination. The Polyp Trap has one chamber to collect the polyps and one open area for straight suction.

The single use Polyp Trap consists of:

- One chamber with one filtered trap to collect polyps, and one open area for straight suction.
- A lid with one tube for connection to the endoscope.
- A polyp removal hollow stick, Please take the stick out before use the polyp trap.

WARNINGS AND PRECAUTIONS

Products marked "single-use" are for single-use only. Do not reuse, reprocess, or resterilize single-use products. The materials used in the manufacture of the device may not withstand repeated reprocessing. The device may not perform as intended by the manufacturer if it is reused. This may lead to failure of the device to perform as intended and/or material degradation, which may result in patient injury, illness or death. Reuse, reprocessing, or resterilization may also increase the risk of contamination of the device and/or cause patient infection or cross-infection, including but not limited to, the transmission of infectious disease(s) from one patient to another. Contamination of the device may lead to injury, illness or death of the patient.

Do not disconnect the pre-installed tubing from the polyp trap lid outer port as this may result in damage to the polyp trap lid and may result in exposure to the contents of the trap during removal.

Protective garments (i.e. gloves, protective eyewear, gowns, etc) should be worn during the use of this device to protect against the risk of cross-contamination and injury.

INSTALLING THE POLYP TRAP

Connection:

1. Connect luer lock fitting end of trap to suction system.
2. Connect tube side of trap to endoscope suction fitting.

Capture:


3. Apply suction via the endoscope suction valve to trap polyp.

Polyp Removal and Transport:

4. After the polyp is trapped, remove the trap from the endoscope and suction system.
5. Separate trap by rotating both halves in opposite directions.
6. Remove polyp (if possible use the hollow stick help to remove the polyp) and place in specimen jar with fixative, or place trap in specimen jar and fill with fixative.
7. Transport to pathology.

Product Disposal:

After use, this product may be a potential biohazard. Handle and dispose of the product in accordance with accepted medical practice and applicable local, state and federal laws and regulations.

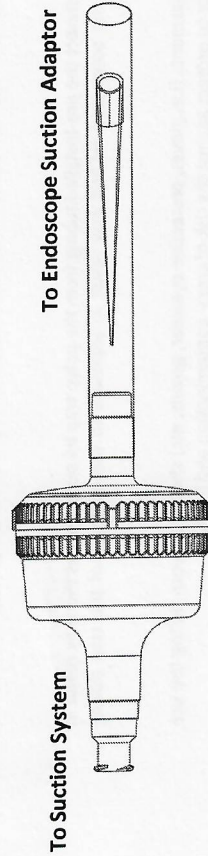

DESCRIPTION OF SYMBOLS USED ON LABELS

	Non-Sterile Device		Instruction for Use
	Do Not Re-use		Quantity in package
	Manufacturing Date		Manufacturer
	Reference No.		Lot Number
	Use by		Keep Dry
	Do Not Contain Phthalates		Prevent from sun light
	Not Made with Natural Rubber Latex		Humidity Limit
	Authorized Representative in the European Community		Temperature Limit
	EC Conformity		Medical Device
	Do not use if package is damaged or open		